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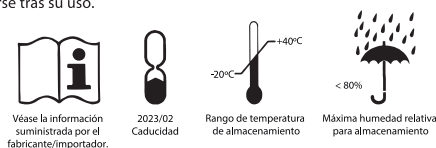
Fabricado por: Chongqing Aibeijian Medical Technology Co., Ltd.  
 No. 12 Shigui Street, Jieshi Town, Banan District, Chongqing, China

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Non-reusable FFP3 filtering mask according to EN 149:2001+A1:2009.

**PERSONAL PROTECTIVE EQUIPMENT.**

This item has been manufactured following the requirements of Regulation (EU) 2016/425 for basic use.

Failure to follow the instructions and warnings on the use of this mask during all times of exposure can reduce the effectiveness of the mask and could result in illness or disability.

As with any respiratory device, the wearer must be adequately trained prior to use. Before use, the wearer should always check the mask is in good condition- no dirt, no damage to headharness attachment, etc.

**Limitation:**

Do not use the respirator or enter or stay in a contaminated area under the following circumstances:

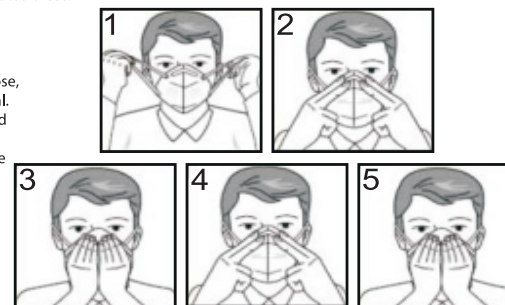
- Atmosphere contains less than 19.5% oxygen.
- If you smell or taste contaminant.
- For protection against gases or vapors.
- Contaminants or their concentrations are unknown or immediately dangerous to life or health.
- For sandblasting, paint-spray operations and asbestos treatment.
- In explosive atmospheres.

**Warnings:**

- This mask marked "NR", shall not be used for more than one shift.
- Never substitute, modify, add or omit parts in the configuration as specified by the manufacturer.
- This mask helps protect against certain particle contaminants but does not completely eliminate exposure to the risk of contracting disease or infection.
- It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.
- Change the mask immediately if breathing becomes difficult or the mask becomes damaged or distorted.
- Change the mask if a proper face seal cannot be achieved.
- Change the mask if it is removed while in the contaminated areas.

**Fitting instructions:**

1. Hold the particle filtering half mask in position over the nose and mouth. Pull the headharness behind to ears.
2. Ensure the nose clip is securely moulded around the nose, resting the ends against the cheek to obtain a good seal.
3. To check a proper fit, cup both hands over the mask and exhale vigorously. If air leaks around the nose, tighten the nose clip, if air leaks around the edge, reposition the headharness for better fit.
4. Repeat adjustments until the mask is sealed properly.
5. If a proper seal cannot be achieved, do not enter the contaminated area or it may cause illness.
6. Recommended use time: no more than 8 hours of continuous use is recommended.


**Storage:**

The mask should not be removed from its package unless it is required and should be discarded after use.

Notified Body: POLSKI REJESTR STATKOW S.A.  
 Al. Gen. Jozefa Hallera 126 80-416 Gdansk, Poland (NB 1463).

Declaration of conformity: It is available at: [www.makito.eu](http://www.makito.eu) (search for item number).

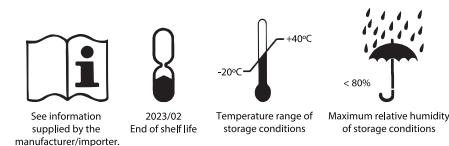Manufactured by: Chongqing Aibeijian Medical Technology Co., Ltd.  
 No. 12 Shigui Street, Jieshi Town, Banan District, Chongqing, China  
 Trade mark: Aibeijian

Imported by: Mtko Catal Importaciones S.L.  
 Ctra. Huércal-Overa S/N 04640 Pulpi, Almería, Spain

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