

5639_PARCHE TERMICO DEBBLY

FRONT PART

5639_Parche Térmico Debbly Manual de instrucciones (Esp)



APLICACIÓN DE FRIO:

CONGELADOR: Introducir en el congelador aproximadamente 1-2 horas para conseguir los mejores resultados.

APLICACIÓN DE CALOR:

Sumergir la bolsa en agua caliente o hirviendo durante un periodo de 10 minutos. Retirar del fuego y dejar enfriar durante 2 minutos, o hasta que alcance una temperatura confortable.

PRECAUCIÓN:

- No utilizar calentadores eléctricos ni microondas para calentar la bolsa.
- No perforar la bolsa.
- Antes de usar, compruebe que la temperatura de la bolsa no es extrema. Se recomienda aplicar envuelto en un paño.

MANTEIMIENTO:

Lavar en agua templada usando jabón neutro.

COMPOSICIÓN:

CMC (5%), Glicerol (10%), Conservante (0.2%) y agua (84.98%).

BACK PART

5639_Hot-Cold Pack Debbly User manual (Eng)



COLD APPLICATION:

FREEZER: Store in the freezer for approximately 1-2 hours before use. When you remove the bag from the freezer, the balls will not be frozen.

HOT APPLICATION:

Fully immerse the gel pack in hot or boiling water for approximately 10 minutes. Remove from heat and let the pack cool down for 2 minutes or until it reaches a comfortable temperature.

CAUTION:

- Do not heat in electric kettle or microwave.
- Do not pierce the pack.
- Before using the hot-cold pack, always check that the temperature is not too extreme against the skin. It is advisable to wrap the pack in a cloth before applying.

CLEANING:

Wash in warm water using neutral soap.

COMPOSITION:

CMC (5%), Glycerol (10%), Preservative (0.2%) & Water (84.98%).